

RE-ORDER CODE

211561

CE
CERTIFIED

P2 DISPOSABLE RESPIRATOR

10 PACK



FFP2 Disposable Respirator is ideal for working environments in which harmful particles can be found in the breathing air such as industries like cleaning, maintenance, construction or for medical professionals to protect themselves against liquid particles, fog, dust, smoke, bacteria and aerosols.

This Disposable folding mask meets the requirements of European Standard EN 149:2001 + A1:2009.

- Colour: White
- Size: one size fits all
- Material: Non-woven
- Latex Free
- Foldable


Features:

- Filtration for very low breathing resistance.
- Protects against hazards, such as, dusts, mists, smoke.
- Lightweight construction for added comfort that may increase wearer time and foldable for convenience
- Two ear loop straps to provide comfortable and secure seal

Time Use Limitation: Replace the respirator when it becomes dirty, damaged, or difficult to breathe through.

STORAGE: Store in a well-ventilated place. Protect from Sunlight. Keep Cool.

FITTING INSTRUCTIONS

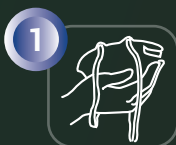


1 Hold respirator in your hands as shown, with the nose piece at your fingertips and the ear loops hanging freely below your hand.




2 Place respirator under your chin with the nosepiece up. Pull the top headband to top back of head.



3 Pull the ear-loop bands over each ear.

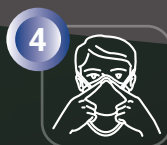

4 Adjust mask for comfortable fit. Push the nose piece inward with your fingertips to conform to the shape of your nose.

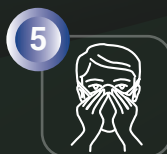

5 Cup both hands over the mask, checking for tightness. Inhale sharply, negative pressure should be felt in the mask, re-adjust as necessary.

Caution:

The respirator should be fit checked for each wearing. Place both hands completely over the respirator and exhale. If air leaks around nose or the edges, adjust the nosepiece and/or earloops until a good fit is achieved.